



# Bio-SkinG

## Clinical study on Burns



## Thermal Wounds – case 1 (1) Female, 55 years old,

### Brief:

- The burning wound was caused by fire on the right leg of the patient.
- With the help of BioSkinG, the wound was healed fully in 16 days and all the BioSkinG fell off in the following week.
- Despite of the exudates, there was no dressing change required during the full treatment.

# Thermal Wounds – case 1 (2)

Female, 55 years old,



**The first day**



**The first day of use**



**4<sup>th</sup> days**



**16<sup>th</sup> days**



## Thermal Wounds – case 2 (1)

Male, 2 years old,



### Brief:

- The II degree burning wound ( 7.5” x 2”) was caused by hot water on the back of the baby.
- A clean wound as it was, with the help of BioSkinG, the wound healed fully in 7 days and all the BioSkinG fell off in 10 days.
- Minimal exudates was observed, no dressing change required during the full course. Hence, the boy suffered no pain.

# Thermal Wounds – case 2 (1)

Male, 2 years old,



**The first day**



**The first day of use**



**7<sup>th</sup> days**



**10<sup>th</sup> days**



# Thermal Wounds – case 3 (1)

Male, 46 years old,



## Brief:

- The II degree burning wound was caused by hot fluid on the left arm of the patient.
- A clean wound as it was, with the help of BioSkinG, the wound healed fully in 9 days and all the BioSkinG was removed by medical personnel on the 9th day.
- No dressing change required during the full healing course. Hence, the patient suffered no pain.

# Thermal Wounds – case 3 (1)

Male, 46 years old,



Day 1 bfr treatment



Day 1 aft treatment



Day 7



Day 9



<fell off after fully healed>